



AUTOCLAVE  
**PRIMA**

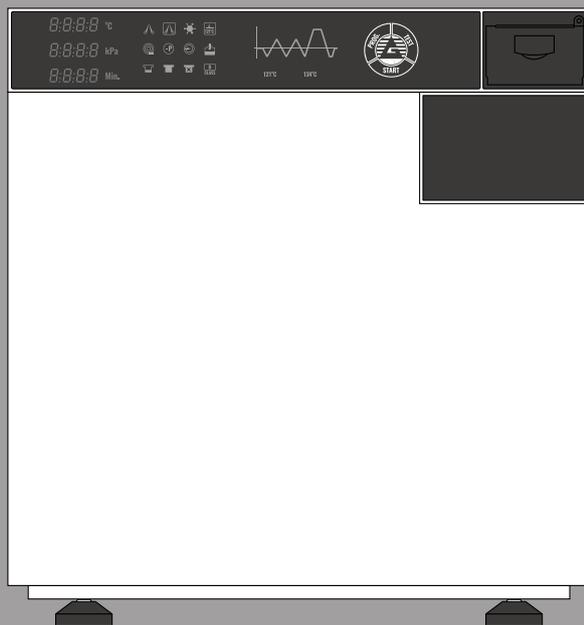


GUIDE FOR QUICK  
INSTALLATION


# AUTOCLAVE PRIMA

1. Please unpack the Prima Autoclave.

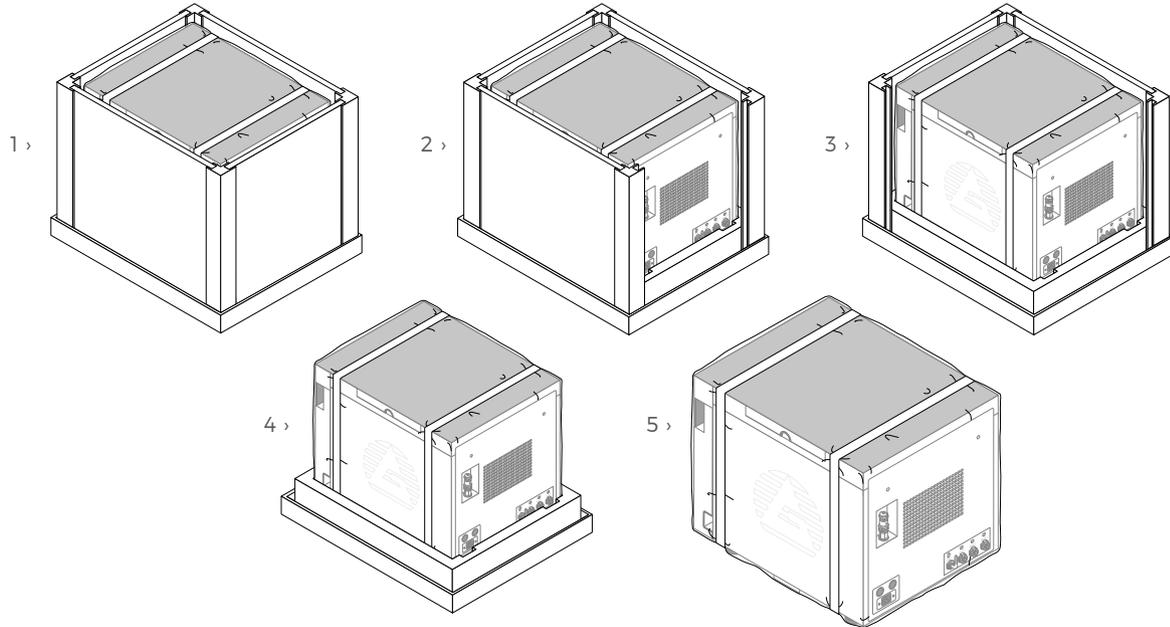








2. Please remove the protective film.

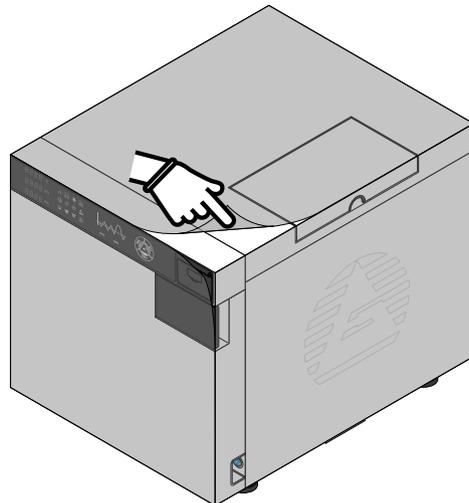

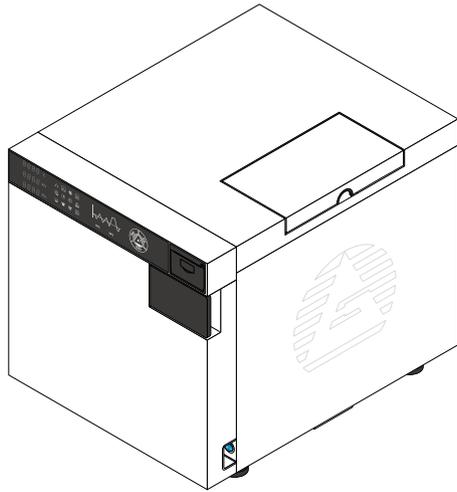

### 3. Check that all accessories are present.

N.	DESCRIPTION	QUANTITY
A	RACK (FOR TRAYS)	1 PIECE
B	TRAYS	4 SET
C	DRAINAGE PIPE	2 PIECES
D	GASKET	1 PIECE
E	INSTRUMENT FOR TRAYS	1 PIECE
F	USER MANUAL	1 PIECE
G	DOOR KEY FOR DOOR REGULATION	1 PIECE
H	USB KEY	1 PIECE
I	SPACERS	2 PIECE

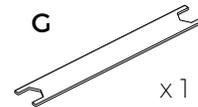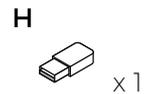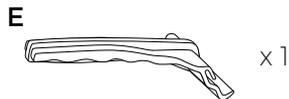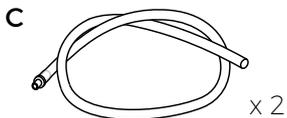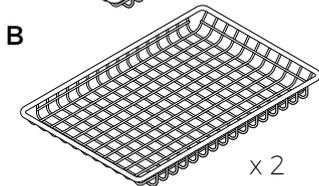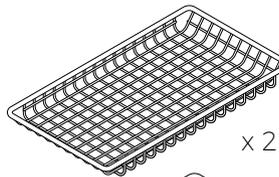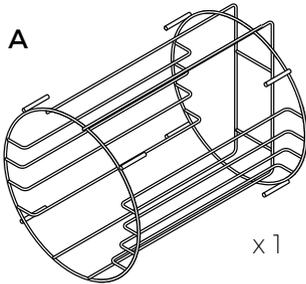

- 4.** Please place the sterilizer on a flat surface area and raise the front-end part 3.5 cm higher than the back-end as illustrated below:


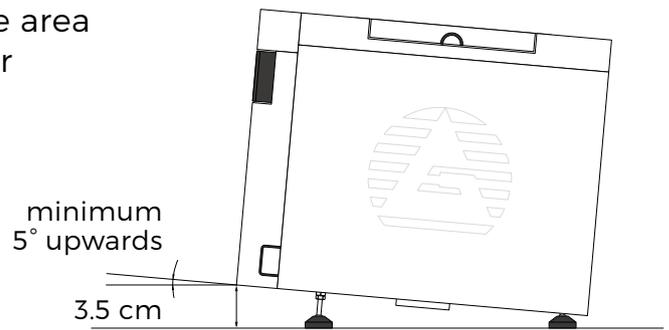


Figure 1

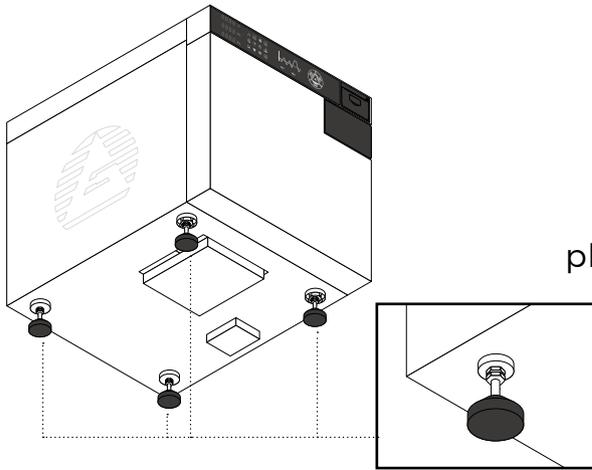

- 5.** To regulate the incline of this sterilizer, please unscrew the nut with the 17 mm key. Please regulate the base and tighten the nut with 17 mm. (Figure 1)

- 6.** Please tighten the spacer on the back end. (Figure 2)


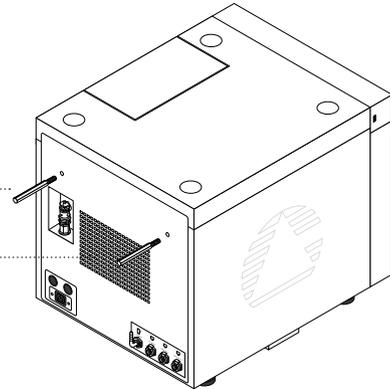


Figure 2

- 7.** Please connect the machine to the power supply. (Figure 3)

Figure 3

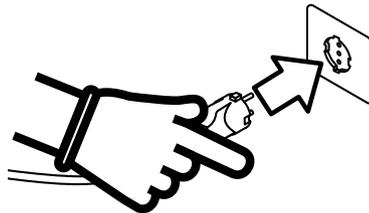

8. Please connect the drain pressure pipe (Connector 1) and the used water pipe (Connector 2) to the available exhaust system through the specific connectors. (Figure 4)



Please remove all accessories from the chamber before switching the machine on.

9. Please switch on the sterilizer through the power switch  located on the bottom right corner of the front side of the sterilizer. (Figure 5)

10. Please set up the machine as follows:

Figure 4




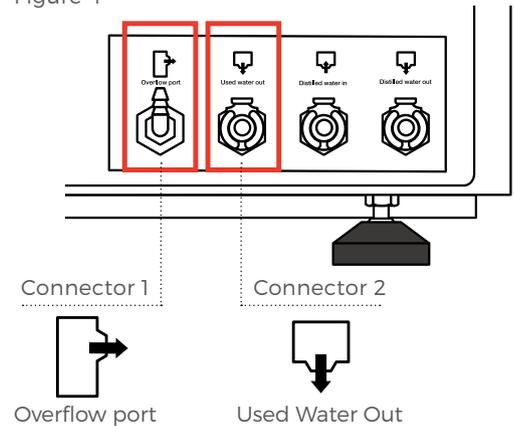


Figure 5

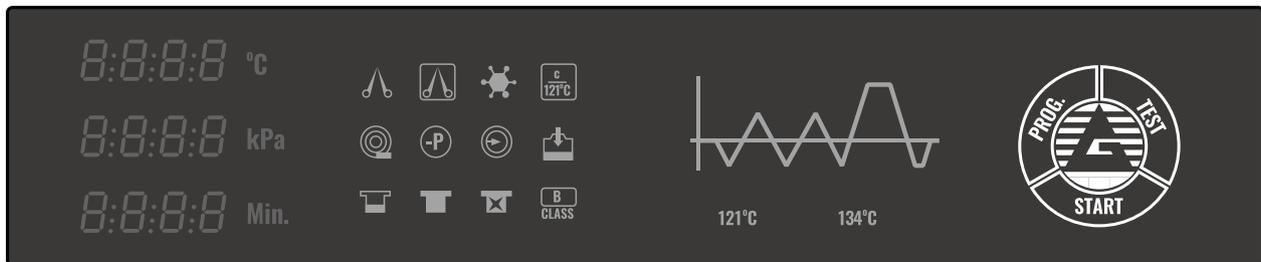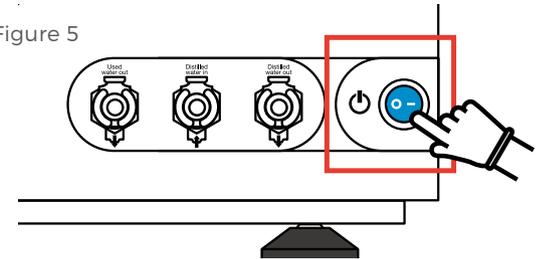


**PROG.** (BUTTON 3)

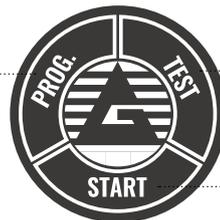

**TEST** (BUTTON 1)

**START** (BUTTON 2)

Figure 6

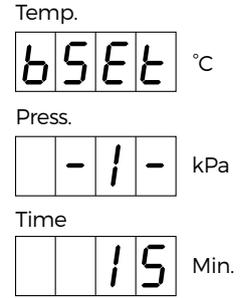


### A. DATE AND TIME SETTING:

- Please turn on **Setting Mode** and keep pressing together to **“PROG”** (Button 3) and **“TEST”** (Button 1) for about 8 seconds. The following graphics will appear on the screen: (Figure 6)
- Please push **PROG.** (Button 3) for choosing the different functions (YY/MM/DD and hh:mm:ss) and **TEST** (Button 1) or **START** (Button 2) for setting the required date and time:

Date and time setting	Year	Month	Day	Hour	Minutes	Second	Exit
STEP:	-1-	-2-	-3-	-4-	-5-	-6-	-7-
DISPLAY:	14	05	24	15	45	55	out

- Once you have set time and date, please push **PROG.** (Button 3) until it appears **“out”** on the screen and please push **START** (Button 2) to exit from **Setting Mode.** (Figure 7a)

Figure 7a

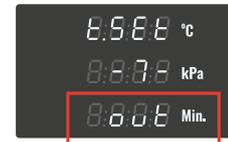

### B. ENERGY-SAVING SETTING:

- Please press **TEST** (Button 1) for about 8 seconds and choose **“POer”** function. When you choose the **“POer”** function, the **“Heat”** function will appear on the screen. (Figure 7b)
- Please press **PROG.** (Button 3) and choose the **“POer”** function. (Figure 7c)
- Please press **TEST** (Button 1) for turning on (0) or off (1) the energy-saving function. (Figure 7d)

Figure 7b



Figure 7c


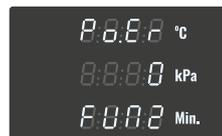


Figure 7d



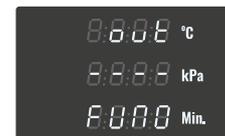

### C. ALTITUDE SETTING:

- Please press together **START** (Button 2) + **TEST** (Button 1) for about 8 seconds.
- T1 function will appear on the screen (Figure 8a).
- Please press **PROG.** (Button 3) for 6 times until “**ALT**” function appears on the screen.
- Please press **TEST** (Button 1) **or START** (Button 2) for increasing or decreasing the altitude value\*\*\* (Figure 8b)
- Please press **PROG.** (Button 3) to exit from the ALT function. (Figure 8c)



\*\*\* Discover more about it:

Visit this site <http://dateandtime.info/it/citycoordinates.php?id=2988507> or download Altimetro Preciso (AR Lab) App.

### D. To check the setting modes, please scroll down with PROG. button for choosing the different functions and with TEST/START buttons for turning them on or off:

- **PRINTER:** ON/OFF (Figure 9a)
- **USB:** ON/OFF (Figure 9b)
- **DRY:** 1 default setting mode (Figure 9c)
- **BLUETOOTH:** ON/OFF (Figure 9d)

To modify these functions:

- Please press **TEST** (Button 1) for about 8 seconds;
- Please press **PROG.** (Button 3) for choosing the several functions (PRINTER, USB, DRY, BLUETOOTH);



- Please press **PROG.** (Button 3) to exit from the Setting Mode and press **START** (Button 2)

**E.** Please install the digital report program according to the technical manual. (Pages 46-48)

**F.** Please switch on the machine and check all functions by choosing the different programs. (Please select the quick wrapped program for testing the sterilizer)



**G.** At the end of the cycle, please check if the report is correctly printed and it is saved on USB.



**FOR FURTHER INFORMATION ABOUT THE TECHNICAL INSTALLATION, PLEASE READ THE TECHNICAL MANUAL ISSUED.**



Via Umbria 10 20090 Segrate Milano ITALIA  
info@galbiati.com www.galbiati.com